# Operator's Manual

## **ELAN** STRAIGHT STAIRLIFT **SRE-3050**

01mar2021  
P/N 3050-O

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# PRODUCT REGISTRATION FORM

The image shows a product registration form for Bruno. On the left is a photograph of a white Bruno stairlift installed on a wooden staircase. The form is titled 'PRODUCT REGISTRATION' and includes a 'Made in USA' logo. A large 'SAMPLE' watermark is overlaid diagonally across the form. The form contains the following fields and instructions:

**PRODUCT REGISTRATION**  Made in USA

Please fill out and return this card or register online.

First Name: \_\_\_\_\_ Last: \_\_\_\_\_

Address: \_\_\_\_\_ City: \_\_\_\_\_

State/Province: \_\_\_\_\_ Zip/Postal Code: \_\_\_\_\_

Phone: (\_\_\_\_) \_\_\_\_\_ Email Address: \_\_\_\_\_

Date of Purchase (m/dd/yyyy): \_\_\_\_\_

Rate your Bruno dealer:

excellent  good  average  poor

Rate your Bruno product:

excellent  good  average  poor

Register online at [bruno.com/warranty](http://bruno.com/warranty)

Bruno is pleased to provide you with this mobility-enhancing product.

The Product Registration Form is shipped in a plastic bag with this manual. It is very important that you fill out and return this form at your earliest convenience so that we may complete the warranty registration process for your unit.

The **serial number** is found on labels affixed to the manual, as well as on the unit itself.

***Serial and model numbers must be provided*** when filing a warranty claim, requesting service or ordering parts. We encourage you to keep this information readily available at all times.

### **Please Note:**

**The warranty for the SRE-3050 Stairlift is rendered null and void if the unit is installed or maintained by anyone other than an authorized Bruno dealer.**

Regulatory Information and Specifications . . . . .4  
Safety. . . . .5  
Circuit Breaker. . . . .6  
Seating. . . . .6  
Operation . . . . . 7-12  
    Standard Swivel Seat . . . . .7  
    **Optional** Power Swivel Seat . . . . .8  
    **Optional** Power Footrest. . . . .9  
    **Optional** Manual Folding Rail . . . . .10  
    **Optional** Power Folding Rail . . . . . 11-12  
Battery Charger . . . . .13  
    Charging . . . . .13  
    LEDs. . . . .13  
Transmitters . . . . .14  
Maintenance . . . . .15  
Troubleshooting . . . . .16  
Vacation and Long-Term Storage . . . . .17  
Warranty. . . . .18

# REGULATORY INFORMATION and SPECIFICATIONS

This lift is intended to be installed in accordance with the current edition of:

- EN 81-40:2008 Safety rules for the construction and installation of lifts - Special lifts for the transport of persons and goods - Part 40: Stairlifts and inclined lifting platforms intended for persons with impaired mobility
- ASME A18.1:2017 Safety Standard for Platform Lifts and Stairway Chairlifts
- CAN/CSA-B613-00 (R2012) - Private Residence Lifts for Persons With Physical Disabilities
- ASME A17.5/CSA B44.1 - 2019 Elevator and Escalator Electrical Equipment
- NFPA 70: National Electrical Code®
- CSA C22.1 Canadian Electrical Code

Verify applicable local codes with the appropriate regulatory authorities.

## SPECIFICATIONS


<i>Weight Capacity:</i>	300 lbs. (136 kg)
<i>Control:</i>	constant pressure (armrest and two remotes)
<i>Maximum Incline:</i>	45 degrees, specials to 52 degrees
<i>Seat Swivel:</i>	0, 67 and 90 degrees at top landing only
<i>Battery Charger:</i>	120VAC, 60 Hz, 0.35A input; 24VDC, 1A output, continuous monitoring

### The SRE-3050 Elan is NOT recommended for these type of users:

- Those who use a scoot-pivot transfer from a wheelchair without weight bearing through their legs, whose full body weight is placed on a single stairlift armrest.
- Those with leg weakness, during stand to sit transfer, whose knees buckle with uncontrolled lowering to the stairlift seat.

For the above type of users, the SRE-2010 Elite IS recommended.

## COMPONENTS OF THE SRE-3050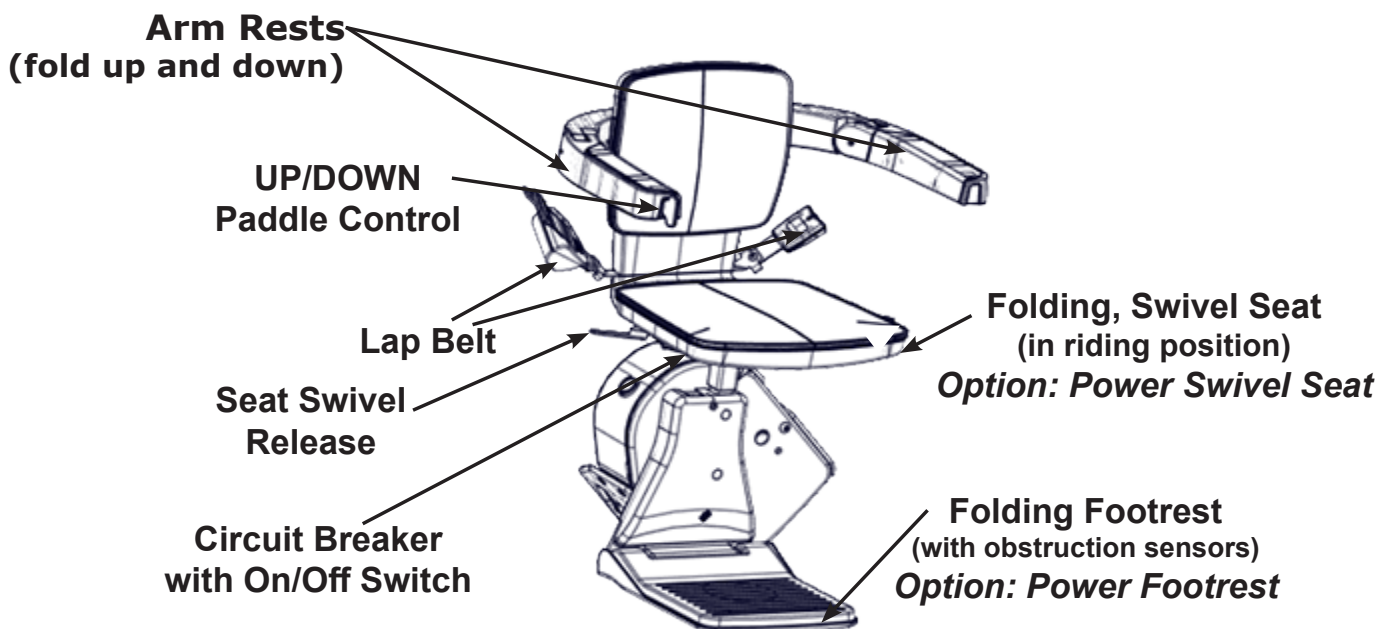

## **FOR YOUR SAFETY**

- Read and understand the operating instructions, and all safety precautions prior to using the SRE-3050.
- Be sure stairs are clear of objects before operating the stairlift.
- Always be aware of the surface onto which you are stepping as you get on and off the stairlift.
- Avoid carrying items with you on the stairlift which could hinder movement of the stairlift up and down the stairs.
- Do not wear LOOSE clothing that could interfere with stairlift movement.
- Fasten your lap belt before operating the stairlift.
- Make sure the seat is locked in the correct riding position.
- Be sure that the seat swivel path is clear before swiveling in or out of the ride position, and before repositioning the seat.
- Be certain the seat swivel is locked in place before attempting to get on or off the stairlift.
- Make certain that there are no obstacles that would prevent complete raising/lowering of the optional power footrest (if unit is so equipped).
- NON-AMBULATORY USERS: Bruno recommends carrying a cellular or cordless telephone or an emergency communication device when riding the stairlift while alone in the house.

Each stairlift is designed to travel down the stairway at a slightly slower speed than is the case when travelling UP.

The footrest is equipped with *obstruction sensors* which stop the carriage if an obstacle is encountered on the stairs.

If this should occur, press the opposite side of the UP/DOWN paddle control (on the armrest) to run the carriage in the opposite direction. Clear the obstacle from the stairway or rail then resume travel in the desired direction.

Your SRE-3050 is battery powered and will continue to operate for some time during a power outage. The length of operation will depend on the battery condition, the length of the stairs and the weight of the rider.


*Only in the event of a prolonged power outage (more than one hour) will it be necessary to turn the power off (circuit breaker switch on the carriage) and turn off or unplug the charger from the wall outlet to conserve battery charge.*

The ***power swivel seat option*** is equipped with current overload sensing to prevent further seat swivel if a jam is detected.

The ***power footrest option*** is equipped with current overload sensing that will cause the footrest to return to its last position if a jam is detected.

The ***power folding rail option*** is equipped with current overload sensing to prevent damage if an obstruction or jam is detected.

## CIRCUIT BREAKER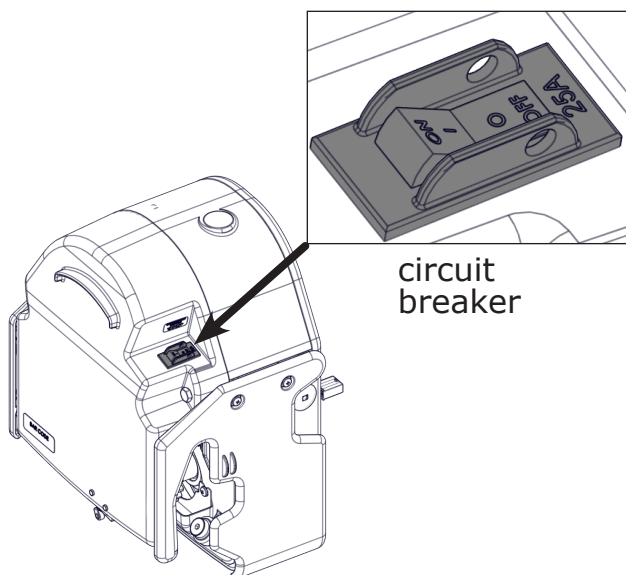

The SRE-3050 includes a circuit breaker with a built-in ON/OFF switch located on the back side of the carriage.


The circuit breaker protects the battery, controller and motor circuits in the stairlift carriage.

It is unlikely that this circuit breaker will trip during normal use. However, if the stairlift becomes inoperative, check the circuit breaker as a first troubleshooting step. Reset it, if necessary.

Possible causes of circuit breaker tripping:

- foreign object jamming the rail or gear rack;
- exceeding the rated weight capacity of the stairlift.

## SEATING

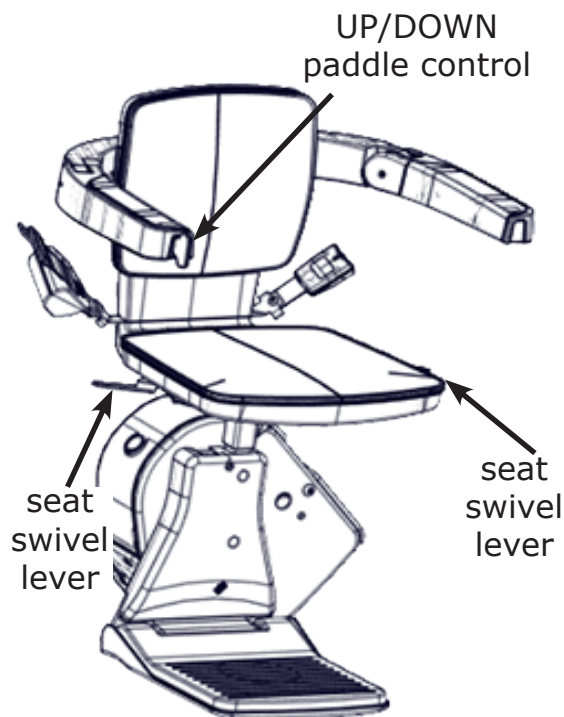The seat assembly on the SRE-3050 is designed to swivel and to lock at 0, 67 and 90 degrees.

Pushing down on either side of the swivel lever allows you to swivel the seat to a convenient position for transferring into or out of the seat.

Once you release the swivel lever, the seat will lock at the next available position.

To lower the seat assembly from a folded position to the riding position, simply push down on the seat.

## **STANDARD SWIVEL SEAT**

The SRE-3050 includes a "**soft-start**" feature which prevents the unit from moving abruptly the instant the UP/DOWN paddle control is pressed.

Press and hold the paddle control for a moment to begin the gentle acceleration of the stairlift to operating speed.

Depressing and holding the paddle control will cause the carriage to move until the carriage stops automatically at the upper or lower end of the stair rail, or until the switch is released.

Two (2) remotes enable you to move the carriage to or from either end of the stair rail.

Remotes are designed to call / send an **empty seat only!** The remotes are NOT designed to move a seat with a person on board.

1. Position yourself in the seat.
2. Fasten the lap belt.
3. Depress and hold the seat swivel lever.
4. Rotate the seat so that you are at right angles to the rail (with your back to the stairway wall).
5. Release the seat swivel lever to secure the seat in that position.  
A safety switch prevents operation of the SRE-3050 unless the seat is locked in position at right angles to the rail (the "riding" position).
6. Lower both armrests completely.
7. Position both feet securely on the footrest and place your arms on the armrests.
8. Press and hold the paddle control (located on the front edge of one of the armrests) to travel UP / DOWN the rail.
9. Once you have reached the top or bottom landing, release the paddle control.
10. Depress and hold the seat swivel lever as you rotate to the position for transferring from the seat.  
*NOTE: The seat will only swivel at the top landing.*
11. Release the seat swivel lever and allow the seat to lock in this position.
12. Unbuckle the lap belt and carefully exit the seat.

### POWER SWIVEL SEAT (optional)

1. Position yourself in the seat.
2. Fasten the lap belt.
3. Position both feet securely on the footrest and place your arms on the armrests.
4. Press and hold the paddle control (located on the front edge of one of the armrests) to travel UP / DOWN the rail.
5. Once you have reached the top or bottom landing, release the paddle control.
6. To rotate the seat **at the top landing**, press the paddle control again on the same side.
7. Release the paddle control when you have fully rotated out.
8. Unbuckle the lap belt and carefully exit the seat.

#### **NOTES:**

- ***You may use the remotes to return the rotated power seat back to the ride position.***
- ***If the unit stops unexpectedly while traveling, check for obstructions or excessive loading. You will only be able to run the unit in the opposite direction until the overload is cleared.***

### POWER FOOTREST (optional)

If your unit is equipped with an optional power footrest, follow the operating instructions indicated below.

1. Raise the seat to raise the footrest to its stored position.
2. Lower the seat to lower the footrest to the ride position.

#### **NOTES:**

- ***Do not try to manually raise or lower the footrest. Always raise/lower the footrest by raising/lowering the seat. Failure to follow this instruction could damage the unit.***
- ***If a jam occurs while raising or lowering the footrest, the footrest will attempt to return to its last position. This will produce a sequence error.***

#### **To restore normal footrest operation:**


- ***Clear the jam.***
- ***Move the seat up and down until the footrest moves again.***
- ***If normal footrest operation is not restored, contact your authorized Bruno dealer.***

## OPERATION: MANUAL folding rail (optional)

When folding the rail UP, the carriage **MUST** be either at the **mid-park** position or parked at the **top** position!

Damage to the carriage **WILL** occur if the carriage is below the mid-park position when the rail is folded up.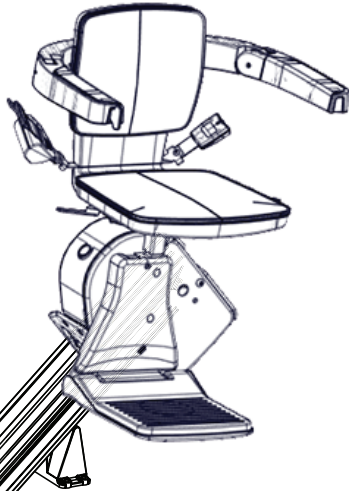


### **WARNING**

#### **PINCH POINT!**

Make sure the areas above and below the folding rail are clear before operating the folding rail.

Keep children and pets at a safe distance.

Failure to observe this warning could result in injury or damage to equipment.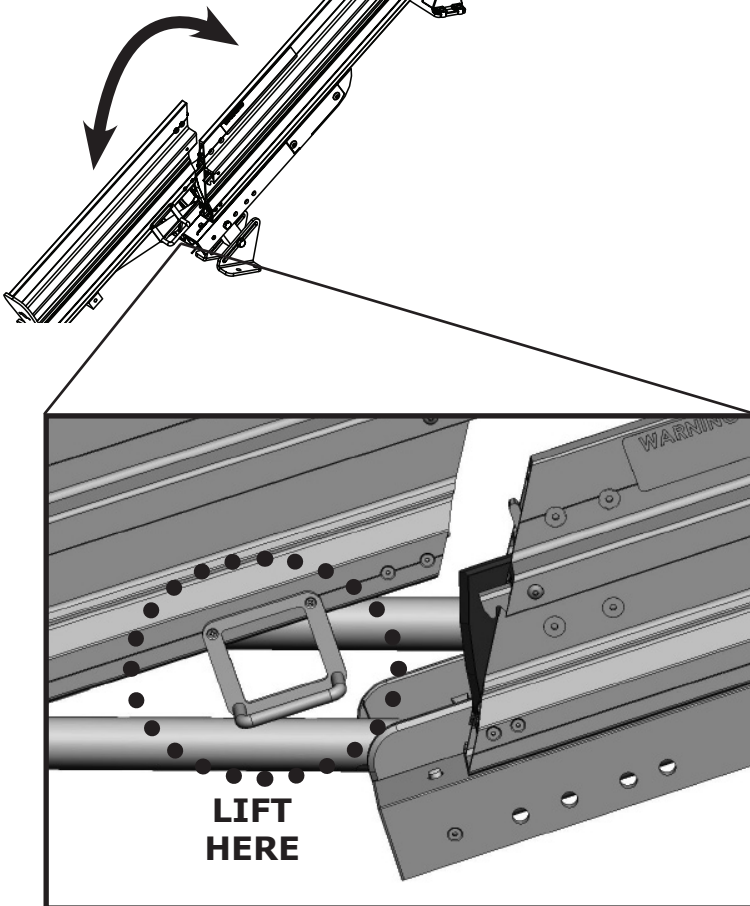

#### **To fold rail UP:**

1. Using the handle, gently pull up and toward the top of the rail.
2. Using the handle, gently lower the folding rail section to the position shown to the right.

#### **To fold rail DOWN:**

1. Using the handle, gently pull up and toward the bottom of the rail.
2. Using the handle, gently lower the folding rail section to the ground.

### **WARNING**

#### **PINCH POINT!**

Make sure the areas above and below the folding rail are clear before operating the folding rail.

Keep children and pets at a safe distance.

Failure to observe this warning could result in injury or damage to equipment.

The **power** folding rail is very similar to the manual folding rail except it has a motorized actuation built into the rail hinge to automatically raise and lower the folding section of the rail.


The power folding rail includes a mid-travel stop position.

- If you are **above** this stop, the rail should always be **folded up**.
- If you are **below** this stop the rail should be **folded down**.

**Do not attempt to manually fold the power rail!**

Using power from the carriage, fold or unfold the rail:

- by riding the stairlift, or
- by using the remotes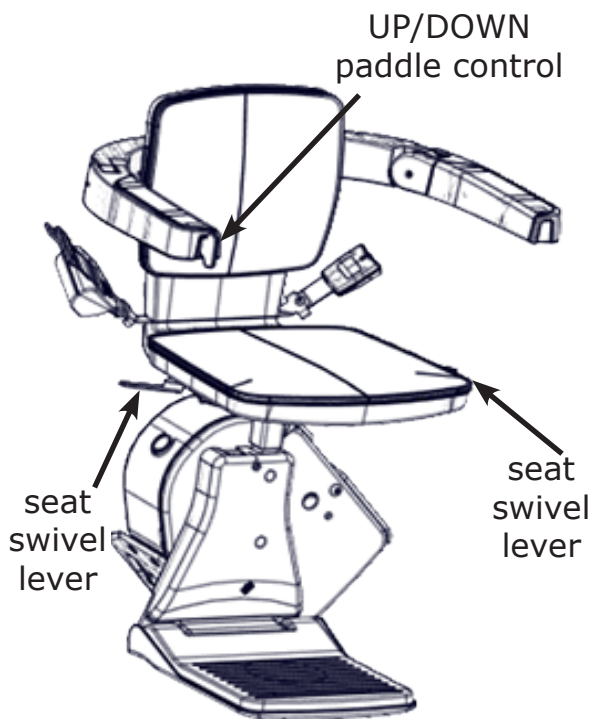

### **Riding the stairlift:**

**If you are at the top of the staircase,** press the UP / DOWN paddle control to ride down the rail until you contact the mid-stop, then release the switch. Look down the staircase to be sure there are no obstructions to prevent the rail from lowering!

Press the UP / DOWN paddle control again, in the down direction. The folding rail will lower. The stairlift will not move down until the rail folds down completely and touches the ground. Release the UP / DOWN paddle control once the rail reaches the ground.

Once the rail is folded down, press and hold the UP / DOWN paddle control again until you reach the bottom of the staircase.

**If you are at the bottom of the staircase,** ride up until you contact the mid stop, then release the UP / DOWN paddle control. Look down the staircase to be sure there are no obstructions to prevent the rail from folding up!

Press the UP / DOWN paddle control again, in the up direction. The rail will rise up. The stairlift will not continue up until the folding rail is completely folded up.

When the rail is in its fully up position, release the UP paddle control, then press it again to travel the rest of the way up the staircase.

### **Note**

You will hear a continuous beep whenever the folding rail is in motion. This is to warn people nearby of rail operation.

In addition, the yellow LED on the control panel flashes.

## OPERATION: POWER folding rail (optional)

### Using the remotes:

**If you are at the bottom of the staircase**, use the remote to power the stairlift down.

**If the stairlift is above the mid-stop position**, the folding rail will not lower until the chair contacts the mid-stop position.

Once the stairlift has reached the mid-stop position, press the remote DOWN button again to unfold the rail.

### **Be sure to stand clear of the rail as it unfolds!**

**If you are at the bottom of the staircase** and want to store the stairlift with the rail folded up, use the remote to send the stairlift up to the mid-stop position. Then press the remote UP button again to raise the rail.

### **Be sure to stand clear of the rail while it folds up!**

The power folding rail is equipped with safety features to reduce the chance of injury to people and pets, as well as damage to property.

If the power folding rail detects an obstruction, the rail will stop and briefly reverse direction.

To resume operation in the desired direction,

1. clear the obstruction,
2. press the armrest switch or remote in the opposite direction,
3. press the armrest switch again, in the desired direction.

### **Disable switch**

The control panel includes a disable switch.


Pressing this switch ON prevents the rail from folding up or down. You would use this function if you wished to keep the rail folded down for an extended period of time.

Be sure to press the switch to OFF allow the rail to fold and unfold correctly under power!

### **IN AN EMERGENCY ONLY!**

Should you need to raise or lower the power folding rail, and you are unable to do so with the UP / DOWN paddle control or the remote, switch the disable switch to OFF. Manually, and GENTLY, raise or lower the rail.

Make sure to lift slowly where indicated by the "LIFT HERE" decal on the short rail. Lifting the rail in a quick, sudden motion may result in damage to the folding rail gearbox.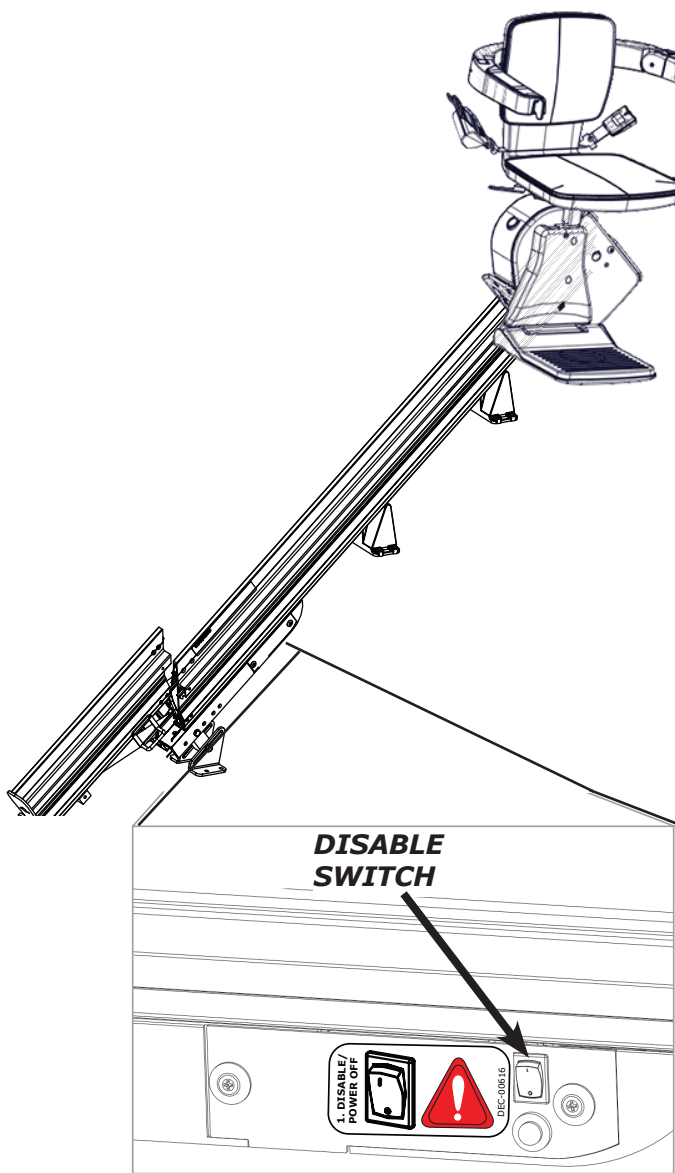

## CHARGING

**Keep the batteries charged** to ensure optimal operation of your SRE-3050.

The batteries are charging whenever the carriage is parked anywhere on the rail.

Failure to keep the batteries charged, or allowing them to fully discharge, will significantly shorten the life of the batteries.

Occasionally check the battery charger LED to make sure the stairlift is being charged correctly.

## LEDS

<b>LED COLOR/ FUNCTION</b>	<b>STATUS/PROBLEM</b>	<b>REMEDY</b>
red	AC power on	normal condition when connected to a live outlet
red (flashing)	defective batteries in unit	Contact your authorized Bruno dealer for service.
yellow	charging	status OK
green	battery fully charged/float mode charging	status OK

## REMOTES

---

The SRE-3050 hand-held remote may experience certain types of interference caused by portable and mobile radio frequency (RF) communications equipment. However, the SRE-3050 uses infrared (IR) communications to minimize the possibility of interference.

Also, receivers are mounted on both sides of the SRE-3050 carriage to minimize interference.

Should interference occur, the unit will stop. This feature has been integrated into the SRE-3050 to ensure your safety.

The direct line between the remote to either of the two (2) remotes should be clear of obstacles for optimal operation. It may be necessary to reposition the remotes so that they are aimed at the carriage.

To reduce the possibility of interference:


- While riding in the seat, ALWAYS operate the elevator using the UP / DOWN paddle control on the armrest.
- Operating the SRE-3050 with a remote while riding in the seat can lead to signal interference.
- **DO NOT** mount the remotes behind an obstacle such as a rail post or around a corner.
- **DO NOT** allow direct sunlight to shine on the receivers (blinding the receivers on the carriage).
- DO replace remote batteries regularly.
  - Depleted or nearly-depleted batteries alter the effective range of the remote.
- DO keep the remote and receiver lens free of dirt and debris.
  - Use a non-abrasive cleaner suitable for glass or acrylic surfaces.
  - Do not use polishes or cleaning products containing wax. These products will leave a film on the lens that will reduce the signal transmission range.

### Remote Battery Replacement

To replace the batteries in the remotes, remove the access panel on the back of the remote.

Two (2) AA **alkaline** batteries required.

***Change batteries annually.***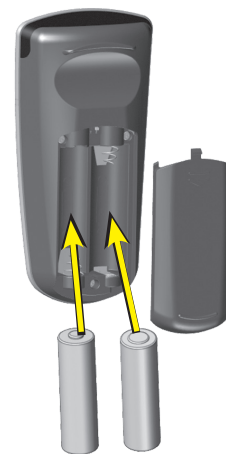

The SRE-3050 is designed to provide many years of reliable service with minimal maintenance.

For best service life:

- Keep the rail channel clean and free of debris.
- Clean all rail surfaces, except gear rack, with Pledge® lemon polish or similar wax-based cleaner once per month.
- Keep the battery charger plugged into a live outlet at all times.
- **At least once a year**, have the stairlift examined, cleaned and lubricated by a qualified, authorized Bruno dealer.

### Seat Upholstery Cleaning

All seating and operator contact areas are fabricated of common materials without a history of causing operator reactions or irritations. The SRE-3050 has been designed assuming that the operator will be wearing clothing covering the torso and footwear. If the user has special sensitivities, normal clothing precautions are suggested.

*For day-to-day cleaning*, use a mild soap and water solution and a soft, lint-free cloth. Do not use vinyl conditioners or protectants.

*For more difficult stains*, use an all-purpose cleaner such as Formula 409® or Fantastik® and a damp, soft cloth or sponge. Rinse with clean water and dry with a lint-free cloth or towel.

## TROUBLESHOOTING


Your SRE-3050 is not operating correctly. What should you do?

Before calling your Bruno dealer, refer to the solutions listed below. This could save you an unnecessary service call!

- **Seat swivel.** Is the seat locked in the riding position?
- **Power seat swivel.** Did the power seat swivel fail to operate properly?

If so:

- Pivot the access panel out.
- Pull the quick-release pin.
- Manually swivel the seat back to the riding position.
- From the bottom, insert the quickrelease pin into the hole closest to the pivot pin. The quick-release pin should pass through the hole in the offset swivel to lock the seat in position.
- Pivot the access panel back to its closed position.
- Contact your authorized Bruno service technician.

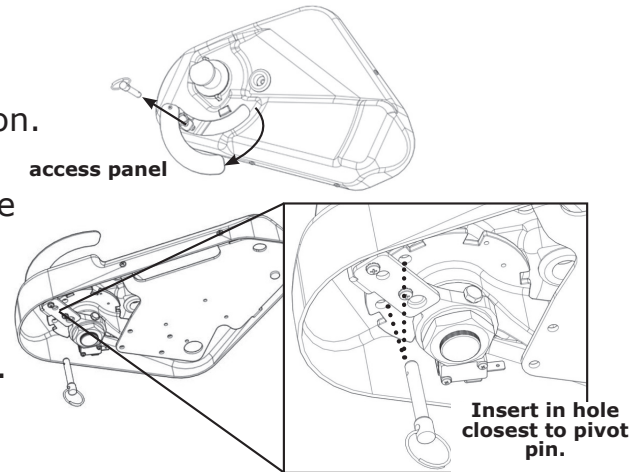- **Footrest.** Is there an obstruction on the stairway that the footrest is contacting?
- **Carriage safety panels.** Is there an obstruction on the stairway that the carriage safety panels are contacting?
- **Main circuit breaker (or fuse) panel of residence.** Has a breaker tripped or a fuse blown?

- **Battery charger.** A discharged battery will cause the carriage to move more slowly until the voltage drops to the point where the controller will shut off. If this should occur, pause for a moment, then run the carriage down to exit the seat.

*Partially charged batteries will run the carriage down, but not up.*

Confirm that the charger is plugged into a live outlet and wait for the batteries to charge. Red and green LED's are illuminated.

Is the battery charger plugged into a functional outlet? *Do not use outlets which are controlled by a wall switch.*

Is the RED power light on?


Has the circuit breaker of the residence tripped? Has a fuse blown in the electrical panel of the residence?

- **Visual check.** Do you notice anything about your stairlift that is different or out of place?

If your SRE-3050 continues to function improperly or not at all after completing the preceding checklist, contact an authorized Bruno dealer to perform all required service, repair and maintenance operations.

## VACATION AND LONG-TERM STORAGE

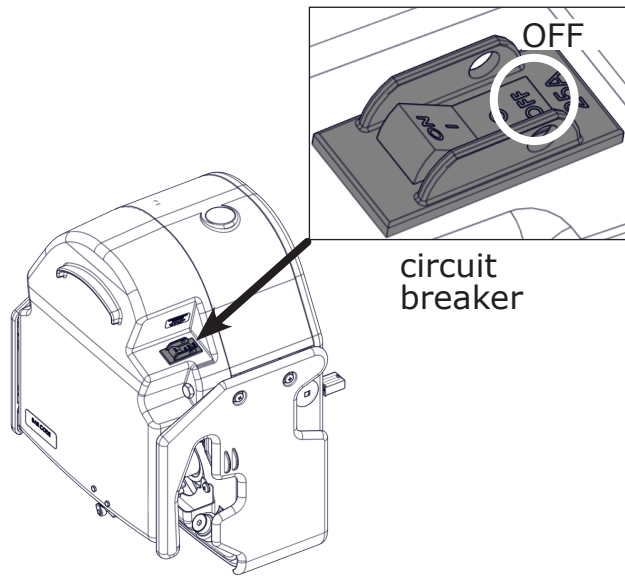If the stairlift will remain unused for an extended period of time, turn off the circuit breaker (ON/OFF switch on the carriage), turn off the battery charger power switch (if equipped) and unplug the charger from the wall outlet.

***The batteries may require recharging before normal use if the stairlift remained in the OFF position for an extended period of time.***

***To recharge, turn on the circuit breaker, reconnect the charger to the wall outlet, and turn on the battery charger power switch (if equipped). Then wait until the charger LED's indicate that the batteries are fully recharged.***

## **BRUNO GOLD LIMITED WARRANTY FOR STAIRLIFT Limited Lifetime Major Components — 2 Year Parts**

Bruno Independent Living Aids, Inc. (Bruno) is confident in its USA-designed and built stairlift units and is pleased to provide the Original Purchaser of a Bruno Stairlift with the following "Industry Leading" warranty. The quality of parts used, along with the care of Bruno's employees in constructing Bruno's products, allow Bruno to provide the following warranty coverage to the Original Purchaser of the Bruno Stairlift:

- ◇ All Parts, other than Major Components, are/will be free of defects in material and workmanship for a period of two (2) years after the date of installation by a Bruno Authorized Dealer.
- ◇ All Major Components, which include the motor, gearbox drivetrain and stationary rail system of the Bruno Stairlift, are/will be free of defects in material and workmanship for the lifetime of the Bruno Stairlift and as long as the Original Purchaser owns the Stairlift. Major Components do not include optional equipment.
- ◇ All labor, to be provided by a Bruno Authorized Dealer, to address any defect of the Bruno Stairlift is covered for the first thirty (30) days after installation thereof by a Bruno Authorized Dealer.
- ◇ All Bruno warranties begin on the date of installation of the Bruno Stairlift by a Bruno Authorized Dealer but that date should be no later than 14 months after it was shipped from the factory.

Bruno Authorized Dealers are trained to evaluate your unique situation and then install the best Bruno Stairlift to meet your needs. The trained installation and service technicians employed by a Bruno Authorized Dealer have the appropriate equipment and resources needed to correctly install and maintain the Bruno Stairlift and should be the first contact for any warranty or service needs.

### **WHAT IS NOT WARRANTY COVERAGE:**

- ◇ Damage to units caused by misuse, abuse, accidents, neglect or modifications NOT made by a Bruno Authorized Dealer.
- ◇ Consumable items such as remote batteries
- ◇ All labor by a Bruno Authorized Dealer after the first thirty (30) days since installation.
- ◇ Reasonable wear and tear.

### **ADDITIONAL NOTES:**

- ◇ All Service and Warranty must be brought to the attention of a Bruno Authorized Dealer. To ensure warranty coverage all maintenance and repair work on the Bruno Stairlift, other than maintenance provided by the Original Purchaser as called for in the Owner's Manual, must be performed by a Bruno Authorized Dealer.
- ◇ In no event shall Bruno be responsible for indirect, incidental or consequential damages, whether such damages arise based on claims based on contract, warranty, tort (including negligence, strict liability or product liability).
- ◇ Bruno, at its sole discretion, reserves the right to replace, repair, or, if not commercially practical and/or feasible, to elect to refund the Original Purchaser the purchase price of the Bruno Stairlift.
- ◇ This warranty gives the Original Purchaser specific legal rights for which the Original Purchaser may also have other rights that vary from state to state. Bruno specifically does not authorize any person to extend the time or scope of this warranty.

For further information regarding this limited warranty, please contact Bruno using the contact information shown below:

In North America:

Bruno Independent Living Aids, Inc.®  
Attn: Service Department  
1780 Executive Drive  
Oconomowoc, Wisconsin 53066 United States  
Tel. (toll free ) 1-800-882-8183

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**PRODUCT REGISTRATION**  Made in USA

Please fill out and return this card or register online.

First Name: \_\_\_\_\_ Last Name: \_\_\_\_\_  
Address: \_\_\_\_\_ City: \_\_\_\_\_  
State/Province: \_\_\_\_\_ Zip/Postal Code: \_\_\_\_\_  
Phone: (\_\_\_\_) \_\_\_\_\_ Email Address: \_\_\_\_\_  
Date of purchase (mm/dd/yy): \_\_\_\_\_

Rate your Bruno dealer:  
 excellent  good  average  poor

Rate your Bruno product:  
 excellent  good  average  poor

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## **ELAN** MONTE-ESCALIER DROIT **SRE-3050**

01mars2021  
P/N 3050-O

The image shows a 'PRODUCT REGISTRATION' card for Bruno. It features a photo of a white Bruno stairlift on the left. The card contains a form with the following fields: 'First Name: \_\_\_\_\_ Last: \_\_\_\_\_', 'Address: \_\_\_\_\_ City: \_\_\_\_\_', 'State/Province: \_\_\_\_\_ Zip/Postal Code: \_\_\_\_\_', 'Phone: (\_\_\_\_) \_\_\_\_\_ Email Address: \_\_\_\_\_', and 'Date of Purchase (mm/dd/yy): \_\_\_\_\_'. Below these are two rating sections: 'Rate your Bruno deal' and 'Rate your Bruno product', each with checkboxes for 'excellent', 'good', 'average', and 'poor'. The card also includes the text 'Please fill out and return this card or register online.', 'Register online at bruno.com/warranty', and the Bruno logo with 'Made in USA'.

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© Bruno Independent Living Aids, Inc. 2019